Grin

Instructions for Use

Grin Scope®
Grin Scope® Mini
Grin Scope® LG
Grin Scope® Mini LG
Grin Mobile App
Grin Web App

EN Device Description: Grin Scope®, Grin Scope® Mini, Grin Scope® LG, Grin Scope® Mini LG, Grin Mobile App, Grin Web App

The Grin Scope® is a reusable,

biocompatible plastic dental cheek retractor that attaches to a mobile phone and works with the Grin App to capture intraoral images and video scans.

Grin Scope Mini: Pediatric version with an 18% smaller opening compared to the standard Grin Scope.

Grin Scope LG Series models:

Same general configuration as

the Grin Scope, with distinguishing features:

- Soft, squeezable silicone housing for comfort.
- Finger-press markings to guide proper insertion.
- Wider flange for improved engagement with inner lips.
- Insertable/extractable prism-type Light Guide component to direct illumination into the Scope's chamber when the phone's flash/LED is offset from the camera lens.

The Grin Mobile App (iOS/Android) guides patients through self-scanning and securely transmits images. The Grin Web App allows care providers to view images, monitor treatment, and communicate with patients.

Rx Only Note: U.S. federal law restricts this device to sale by or on the order of a licensed dentist or physician.

Indications for Use

The Grin Scope is intended for capturing intraoral images during dental or orthodontic treatment. It is used by patients or their legal guardians, under the supervision of a licensed care provider, for remote monitoring of treatment progress.

⚠️ Contraindications and Precautions

- Do not use if components are detached, loose, or damaged.
- Do not use for purposes other than intraoral imaging.
- Do not run, walk, or engage in physical activity with the device in the mouth.
- Do not wrap the strap around body parts.
- Do not expose the device to boiling water (risk of warping).
- Avoid abrasive cleaners.
- Keep out of reach of children when not in use.

Choking Hazard (children and pets): Small components may be swallowed or aspirated if detached.

Minimum Requirements

- Compatible with most iOS and Android phones
- Device must have a functioning back-facing camera, flash/LED, and data or Wi-Fi connection.
- Storage requirements are subject to change; confirm via app store before installation.

Operating Instructions

Setup

1. Download the Grin App from the link in your care provider's email/SMS or scan the QR code on the packaging.

For assistance, contact support@get-grin.com.

2. Attach the scope over the back-facing camera of your mobile phone. Tighten the strap to secure.
3. If using the Grin Scope LG model:
 - If the phone's flash/LED light is further from the camera lens area, ensure the Light Guide is inserted, seated and flush with the device phone side device wall
 - Align the Light Guide with the phone's flash/LED.

Scanning

1. Open the Grin App and follow in-app instructions.
2. Position yourself in front of a mirror. Hold your phone horizontally.
3. Place lips over the scope opening.
 - For the Grin Scope LG models – press on the top and bottom finger grips of the Scope to squeeze the mouth side opening for easier insertion into the mouth
4. Tap Start Recording and follow voice instructions.

Handling the Light Guide (LG Models Only)

- To Insert: Place in socket → Slide gently until flush.
- To Remove: Pull lightly with thumb/finger → Pull out to remove.
- Caution: Do not force insertion or removal.

Cleaning and Storage

- Always remove the Light Guide before cleaning.
- Wash the scope after each use:
 - Dishwasher: Top rack only.
 - Handwash: Mild soap + warm water.
- Rinse thoroughly and dry with a clean paper towel.
- Inspect for damage or missing parts before reuse.
- Store in a clean, dry environment.

Reporting and Compliance

Any serious incident related to the device must be reported:

- To Get Grin Inc. (support@get-grin.com)
- To the competent authority of the Member State in which the user resides (if applicable)

Manufacturer Information

support@get-grin.com
+1 646-893-6122

Get Grin Inc, www.get-grin.com
382A Rt 59, Airmont, NY 10952

EU REP **European Authorized Representative:**
Emergo Europe, Westervoortsedijk 60
6827 AT Arnhem, The Netherlands

| | |
|------------|---|
| | Consult instructions for use |
| | Do not use if package is damaged and consult instructions for use |
| | Manufacturer |
| | Patient Information website |
| MD | Medical Device |
| | Packaging unit and quantity |
| REF | Catalog number |
| LOT | Batch code |
| | Date of manufacture |
| UDI | Unique device identifier |

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- À Get Grin Inc. (support@get-grin.com)
Get Grin Inc, www.get-grin.com  382A Rt 59, Airmont, NY 10952